

**Overview****Method Name**

Enzyme Immunoassay (EIA)

**NY State Available**

Yes

**Specimen****Specimen Type**

Plasma EDTA

**Specimen Required****Collection Container/Tube:** Lavender top (EDTA)**Submission Container/Tube:** Plastic vial**Specimen Volume:** 1 mL Plasma**Collection Instructions:**

1. Immediately place specimen on ice.
2. Centrifugation should be performed within 20 minutes of collection at 900 x g for 10 minutes at 4 degrees C.
3. Carefully remove 1 mL of EDTA plasma from the upper part of the tube and place in a plastic vial.
4. Send frozen.

**Specimen Minimum Volume**

Plasma: 0.5 mL

**Reject Due To**

Gross hemolysis	Reject
Gross lipemia	Reject
Specimens other than EDTA plasma	Reject

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Plasma EDTA	Frozen	28 days	

**Clinical & Interpretive****Clinical Information**

Refer to [www.eurofins-viracor.com/test-menu/](http://www.eurofins-viracor.com/test-menu/)

**Reference Values**

0-1.0 ng/mL

**Performance****PDF Report**

No

**Day(s) Performed**

Monday, Tuesday, Thursday

**Report Available**

4 to 7 days

**Performing Laboratory Location**

Eurofins Viracor

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

83088

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FHSPL	Histamine Plasma	2416-6

---

Result ID	Test Result Name	Result LOINC® Value
FHSPL	Histamine Plasma	2416-6