

# **Test Definition: FMPRE**

Maple Red (Acer rubrum) IgE

_		-	
<i>(</i> )\	er/	<b>\/I</b> (	21A1
$\mathbf{v}$	<i>,</i> CI	VIV	3 VV

## **Method Name**

Enzyme Immunoassay (EIA)

#### **NY State Available**

Yes

## **Specimen**

## **Specimen Type**

Serum

## **Specimen Required**

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

## **Specimen Minimum Volume**

0.5 mL

## **Reject Due To**

Hemolysis:	NA
Thawing:	Warm OK; Cold OK
Lipemia:	Mild OK; Gross reject
Icterus:	NA
Other:	NA

## **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	365 days	

## **Clinical & Interpretive**

## **Reference Values**

<0.35 kU/L



## **Test Definition: FMPRE**

Maple Red (Acer rubrum) IgE

## Interpretation

Class	IgE (kU/L)	<u>Comment</u>
0	<0.35	<b>Below Detection</b>
1	0.35-0.69	Low Positive
2	0.70-3.49	<b>Moderate Positive</b>
3	3.50-17.49	Positive
4	17.50-49.99	Strong Positive
5	>=50	Very Strong Positive

#### **Performance**

#### **Method Description**

The conventional EIA uses allergen-coated discs from several suppliers and an enzyme-labeled anti-IgE.

#### **PDF Report**

No

## Day(s) Performed

Monday through Friday

#### Report Available

2 to 7 days

## **Performing Laboratory Location**

**Eurofins Viracor** 

## **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

#### **CPT Code Information**



# **Test Definition: FMPRE**

Maple Red (Acer rubrum) IgE

86003

## **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FMPRE	Maple Red (Acer rubrum) IgE	7479-9

Result ID	Test Result Name	Result LOINC® Value
Z3094	Maple Red (Acer rubrum) IgE	7479-9
Z3095	Class	102427-2