

**Overview****Reflex Tests**

Test Id	Reporting Name	Available Separately	Always Performed
FOVAT	Anti-Ovary Ab Titer	No	No

**Testing Algorithm**

If FOVAS "Ovarian Ab Screen w/Reflex" is positive, then FOVAT "Anti-Ovary Ab Titer" will be performed at an additional charge.

**Method Name**

Immunofluorescence Assay (IFA)

**NY State Available**

Yes

**Specimen****Specimen Type**

Serum

**Specimen Required**

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1 mL of serum refrigerated in a plastic vial.

**Specimen Minimum Volume**

0.3 mL

**Reject Due To**

Hemolysis	Mild OK; Gross reject
Lipemia	Mild OK; Gross reject
Icterus	NA
Other	NA

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
---------------	-------------	------	-------------------

Serum	Refrigerated (preferred)	14 days	
	Ambient	7 days	
	Frozen	30 days	

## Clinical & Interpretive

### Reference Values

Anti-Ovary Antibody: Negative

Anti-Ovary Ab Titer: <1:5

## Performance

### PDF Report

No

### Day(s) Performed

Wednesday

### Report Available

2 to 17 days

### Performing Laboratory Location

Quest Diagnostics Nichols Institute

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### CPT Code Information

86255

86256 (if appropriate)

### LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FOVAS	Ovarian Ab Screen w/Reflex	21436-1

## Test Definition: FOVAS

Ovarian Antibody Screen with Reflex to Titer,  
IFA

Result ID	Test Result Name	Result LOINC® Value
Z3669	Anti-Ovary Antibody	21436-1