

# **Test Definition: FCURV**

Curvularia spicifera/Bipolaris IgE

# **Overview**

### **Method Name**

ImmunoCAP FEIA

#### **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Serum

# **Specimen Required**

Specimen Type: Serum
Container/Tube: Red or SST
Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 0.5 mL

of serum refrigerated in a plastic vial.

## **Specimen Minimum Volume**

0.5 mL

# **Reject Due To**

Hemolysis:	Mild OK; Gross Reject
Thawing:	Warm OK; Cold OK
Lipemia:	Mild OK; Gross Reject
Icterus:	Mild OK; Gross Reject
Other:	NA

# **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	365 days	

# **Clinical & Interpretive**



# **Test Definition: FCURV**

Curvularia spicifera/Bipolaris IgE

#### **Reference Values**

<0.35 kU/L

## Interpretation

Class	IgE (kU/L)	Comment
0	<.10	Negative
0/1	0.10 – 0.34	Equivocal/Borderline
1	0.35 – 0.69	Low Positive
2	0.70 – 3.49	Moderate Positive
3	3.50 – 17.49	High Positive
4	17.50 – 49.99	Very High Positive
5	50.00 – 99.9	Very High Positive
6	>99.99	Very High Positive

#### **Performance**

## **Method Description**

The ImmunoCAP FEIA method uses as the solid phase a flexible, hydrophobic cellulosic polymer to which allergen has been covalently linked. The advantage of this system is that it has a very high antigen binding capacity when compared to other systems and it has minimal non-specific binding with high total IgE. Viracor Eurofins provides an optional low range calibrator at 0.1 kU/L and a 0/1 class.

## PDF Report

No

#### Day(s) Performed

Monday through Friday

## Report Available

1 to 4 days

## **Performing Laboratory Location**

**Eurofins Viracor** 

## **Fees & Codes**

## Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.



# **Test Definition: FCURV**

Curvularia spicifera/Bipolaris IgE

Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

## **Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

### **CPT Code Information**

86003

## **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FCURV	Curvularia spicifera/Bipolaris IgE	42198-2

Result ID	Test Result Name	Result LOINC® Value
Z3813	Curvularia spicifera/Bipolaris IgE	42198-2
Z3814	CLASS	102267-2