

## Overview

**Method Name**

Gas Chromatography/Mass Spectrometry (GC/MS)

**NY State Available**

Yes

## Specimen

**Specimen Type**

Urine

**Specimen Required**

**Container/Tube:** Plastic, preservative-free urine container

**Specimen Volume:** 20 mL

**Collection Instructions:**

1. Collect 20 mL random urine without preservative.
2. Transfer into a preservative-free plastic urine container.
3. Ship refrigerate.

**Specimen Minimum Volume**

10 mL

**Reject Due To**

All specimens will be evaluated by the processing and performing laboratories for test suitability.

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	10 days	
	Ambient	72 hours	
	Frozen	396 days	

## Clinical & Interpretive

**Reference Values**

Results in ng/mL are reported to the limit of quantitation for the analysis.

**Performance****PDF Report**

No

**Day(s) Performed**

Varies

**Report Available**

16 to 20 days

**Performing Laboratory Location**

Medtox Laboratories, Inc.

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**CPT Code Information**

80375

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FMETH	Methaqualone, Confirmation	16244-6

Result ID	Test Result Name	Result LOINC® Value
Z4116	Methaqualone, Urine Quant	16244-6