

Test Definition: XSRM

Additional Sample for Reflex Oligoclonal Banding, Serum

Overview

Method Name

Only orderable as part of a profile. For more information see MSP3 / Multiple Sclerosis (MS) Cascade, Serum and Spinal Fluid.

Collection Only

NY State Available

No

Specimen

Specimen Type

Serum

Specimen Required

Only orderable as part of a profile. For more information see MSP3 / Multiple Sclerosis (MS) Cascade, Serum and Spinal Fluid.

Collection Container/Tube:

Preferred: Serum gel **Acceptable:** Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL Collection Instructions:

- 1. Within 2 hours of collection, centrifuge and aliquot serum in plastic vial.
- 2. Label specimen as serum.

Specimen Minimum Volume

0.5 mL

Reject Due To

Gross	OK
hemolysis	
Gross lipemia	OK
Gross icterus	OK

Specimen Stability Information



Test Definition: XSRM

Additional Sample for Reflex Oligoclonal Banding, Serum

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	14 days	
	Ambient	14 days	
	Refrigerated	14 days	

Clinical & Interpretive

Performance

PDF Report

No

Specimen Retention Time

14 days

Performing Laboratory Location

Mayo Clinic Laboratories - Rochester Superior Drive

Fees & Codes

Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

Test Classification

Not Applicable

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
XSRM	Additional sample for reflex OLIGS	No LOINC Needed

	Result ID	Test Result Name	Result LOINC® Value	
	XSRM	Additional sample for Reflex OLIGS	No LOINC Needed	