

Overview

Useful For

Providing a genetic evaluation for patients with a personal or family history suggestive of Alagille syndrome

Establishing a diagnosis of Alagille syndrome

Genetics Test Information

This test utilizes next-generation sequencing to detect single nucleotide, small deletion-insertion, and copy number variants in two genes (*JAG1*, *NOTCH2*) associated with Alagille syndrome. See [Targeted Genes and Methodology Details for Alagille Syndrome Gene Panel](#) and Method Description for additional details.

Identification of a disease-causing variant may assist with diagnosis, prognosis, clinical management, familial screening, and genetic counseling for Alagille syndrome.

Special Instructions

- [Informed Consent for Genetic Testing](#)
- [Informed Consent for Genetic Testing \(Spanish\)](#)
- [Hereditary Renal Genetic Testing Patient Information](#)
- [Targeted Genes and Methodology Details for Alagille Syndrome Gene Panel](#)

Method Name

Sequence Capture and Next Generation Sequencing (NGS)

NY State Available

Yes

Specimen

Specimen Type

Varies

Ordering Guidance

Targeted testing for familial variants (also called site-specific or known mutations/variants testing) is available for the genes on this panel. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

Customization of this panel and single gene analysis for any gene present on this panel are available. For more information, see CGPH / Custom Gene Panel, Hereditary, Next-Generation Sequencing, Varies.

Shipping Instructions

Specimen preferred to arrive within 96 hours of collection.

Specimen Required

Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. Call 800-533-1710 for instructions for testing patients who have received a bone marrow transplant.

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Any anticoagulant

Specimen Volume: 3 mL

Collection Instructions:

1. Invert several times to mix blood.
2. Send whole blood specimen in original tube. **Do not aliquot.**

Specimen Stability Information: Ambient (preferred)/Refrigerated

Forms

1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file. The following documents are available:

[-Informed Consent for Genetic Testing \(T576\)](#)

[-Informed Consent for Genetic Testing-Spanish \(T826\)](#)

2. [Hereditary Renal Genetic Testing Patient Information \(T918\)](#)

3. [If not ordering electronically, complete, print, and send a Renal Diagnostics Test Request \(T830\)](#) with the specimen.

Specimen Minimum Volume

1 mL

Reject Due To

All specimens will be evaluated at Mayo Clinic Laboratories for test suitability.

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Varies		

Clinical & Interpretive**Clinical Information**

Alagille syndrome (ALGS) is a multisystemic genetic condition with a wide spectrum of clinical variability. Characteristic features of ALGS include bile duct paucity, cholestasis, congenital heart defects, butterfly vertebrae, eye findings, and facial features including a broad forehead, deeply set eyes, and a small, pointed chin.(1) Approximately 40% of individuals also have kidney disease. The kidney features may include structural abnormalities (eg, small hyperechoic kidney, kidney cysts, or ureteropelvic obstruction) as well as kidney dysfunction, such as renal tubular acidosis.

Disease-causing variants in the *JAG1* gene account for approximately 94% to 97% of ALGS, while disease-causing *NOTCH2* gene variants account for approximately 1% to 3% of cases.(1,2) A small percentage of individuals with a clinical diagnosis of ALGS do not have an identified disease-causing variant in either *JAG1* or *NOTCH2*.

The severity of ALGS is highly variable among individuals, ranging from mild and subclinical, to severe heart or liver disease requiring transplantation. ALGS is inherited in an autosomal dominant manner. Approximately 30% to 50% of individuals inherit ALGS from an affected parent, while about 50% to 70% of cases are *de novo*. Parental somatic mosaicism and germline mosaicism have been reported.

Reference Values

An interpretive report will be provided.

Interpretation

All detected variants are evaluated according to American College of Medical Genetics and Genomics recommendations.⁽³⁾ Variants are classified based on known, predicted, or possible pathogenicity and reported with interpretive comments detailing their potential or known significance.

Cautions

Clinical Correlations:

Test results should be interpreted in the context of clinical findings, family history, and other laboratory data. Misinterpretation of results may occur if the information provided is inaccurate or incomplete.

If testing was performed because of a clinically significant family history, it is often useful to first test an affected family member. Detection of a reportable variant in an affected family member would allow for more informative testing of at-risk individuals.

To discuss the availability of additional testing options or for assistance in the interpretation of these results, contact the Mayo Clinic Laboratories genetic counselors at 800-533-1710.

Technical Limitations:

Next-generation sequencing may not detect all types of genomic variants. In rare cases, false-negative or false-positive results may occur. The depth of coverage may be variable for some target regions; assay performance below the minimum acceptable criteria or for failed regions will be noted. Given these limitations, negative results do not rule out the diagnosis of a genetic disorder. If a specific clinical disorder is suspected, evaluation by alternative methods can be considered.

There may be regions of genes that cannot be effectively evaluated by sequencing or deletion and duplication analysis as a result of technical limitations of the assay, including regions of homology, high guanine-cytosine (GC) content, and repetitive sequences. Confirmation of select reportable variants will be performed by alternate methodologies based on internal laboratory criteria.

This test is validated to detect 95% of deletions up to 75 base pairs (bp) and insertions up to 47 bp. Deletions-insertions (delins) of 40 or more bp, including mobile element insertions, may be less reliably detected than smaller delins.

Deletion/Duplication Analysis:

This analysis targets single and multi-exon deletions/duplications; however, in some instances single exon resolution cannot be achieved due to isolated reduction in sequence coverage or inherent genomic complexity. Balanced structural rearrangements (such as translocations and inversions) may not be detected.

This test is not designed to detect low levels of mosaicism or to differentiate between somatic and germline variants. If there is a possibility that any detected variant is somatic, additional testing may be necessary to clarify the significance of results.

Genes may be added or removed based on updated clinical relevance. Refer to the [Targeted Genes and Methodology Details for Alagille Syndrome Gene Panel](#) for the most up to date list of genes included in this test. For detailed information regarding gene specific performance and technical limitations, see Method Description or contact a laboratory genetic counselor.

If the patient has had an allogeneic hematopoietic stem cell transplant or a recent blood transfusion, results may be inaccurate due to the presence of donor DNA. Call Mayo Clinic Laboratories for instructions for testing patients who have received a bone marrow transplant.

Reclassification of Variants:

Currently, it is not standard practice for the laboratory to systematically review previously classified variants on a regular basis. The laboratory encourages healthcare providers to contact the laboratory at any time to learn how the classification of a particular variant may have changed over time.

Variant Evaluation:

Evaluation and categorization of variants are performed using published American College of Medical Genetics and Genomics and the Association for Molecular Pathology recommendations as a guideline.⁽³⁾ Other gene-specific guidelines may also be considered. Variants are classified based on known, predicted, or possible pathogenicity and reported with interpretive comments detailing their potential or known significance. Variants classified as benign or likely benign are not reported.

Multiple in silico evaluation tools may be used to assist in the interpretation of these results. The accuracy of predictions made by in silico evaluation tools is highly dependent upon the data available for a given gene, and periodic updates to these tools may cause predictions to change over time. Results from in silico evaluation tools are interpreted with caution and professional clinical judgement.

Rarely, incidental findings or secondary findings may implicate another predisposition or presence of active disease. Incidental findings may include, but are not limited to, results related to the sex chromosomes. These findings will be carefully reviewed to determine whether they will be reported.

Clinical Reference

1. Spinner NB, Gilbert MA, Loomes KM, Krantz ID: Alagille syndrome. In: Adam MP, Ardinger HH, Pagon RA, et al, eds. GeneReviews [Internet]. University of Washington, Seattle; 2000. Updated December 12, 2019. Accessed June 7, 2022. Available at www.ncbi.nlm.nih.gov/books/NBK1273/
2. Turnpenny P, Ellard S: Alagille syndrome: pathogenesis, diagnosis and management. *Eur J Hum Genet.* 2012 Mar;20(3):251-257
3. Richards S, Aziz N, Bale S, et al: Standards and guidelines for the interpretation of sequence variants: a joint consensus recommendation of the American College of Medical Genetics and Genomics and the Association for Molecular Pathology. *Genet Med.* 2015 May;17(5):405-424

Performance

Method Description

Next-generation sequencing (NGS) and/or Sanger sequencing are performed to test for the presence of variants in coding regions and intron/exon boundaries of the genes analyzed, as well as some other regions that have known pathogenic variants. The human genome reference GRCh37/hg19 build was used for sequence read alignment. At least 99% of the bases are covered at a read depth over 30X. Sensitivity is estimated at above 99% for single nucleotide variants, above 94% for deletions-insertions (delins) less than 40 base pairs (bp), above 95% for deletions up to 75 bp and insertions up to 47 bp. NGS and/or a polymerase chain reaction-based quantitative method is performed to test for the presence of deletions and duplications in the genes analyzed.

There may be regions of genes that cannot be effectively evaluated by sequencing or deletion and duplication analysis as a result of technical limitations of the assay, including regions of homology, high guanine-cytosine (GC) content, and repetitive sequences. See [Targeted Genes and Methodology Details for Alagille Syndrome Gene Panel](#) for details regarding the targeted genes analyzed for each test and specific gene regions not routinely covered. (Unpublished Mayo method)

Confirmation of select reportable variants may be performed by alternate methodologies based on internal laboratory criteria.

Genes analyzed: *JAG1*, *NOTCH2*

PDF Report

Supplemental

Day(s) Performed

Varies

Report Available

28 to 42 days

Specimen Retention Time

Whole blood: 2 weeks (if available); Extracted DNA: 3 months

Performing Laboratory Location

Rochester

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Mayo Clinic in a manner consistent with CLIA requirements. It has not been cleared or approved by the US Food and Drug Administration.

CPT Code Information

81407
81479
81479 (if appropriate for government payers)

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
ALAGP	Alagille Syndrome Gene Panel	51966-0

Result ID	Test Result Name	Result LOINC® Value
618031	Test Description	62364-5
618032	Specimen	31208-2
618033	Source	31208-2
618034	Result Summary	50397-9
618035	Result	82939-0
618036	Interpretation	69047-9
618037	Additional Results	82939-0
618038	Resources	99622-3
618039	Additional Information	48767-8
618040	Method	85069-3
618041	Genes Analyzed	48018-6
618042	Disclaimer	62364-5
618043	Released By	18771-6