

## Overview

### Useful For

Classification of lymphomas

### Reflex Tests

| Test Id | Reporting Name            | Available Separately | Always Performed |
|---------|---------------------------|----------------------|------------------|
| IHTOI   | IHC Initial, Tech Only    | No                   | No               |
| IHTOA   | IHC Additional, Tech Only | No                   | No               |

### Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

### Method Name

Immunohistochemistry (IHC)

### NY State Available

Yes

## Specimen

### Specimen Type

TECHONLY

### Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

### Shipping Instructions

Attach the green "Attention Pathology" address label (T498) and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

### Specimen Required

**Specimen Type:** Tissue

**Supplies:** Immunostain Technical Only Envelope (T693)

**Container/Tube:** Immunostain Technical Only Envelope

**Preferred:**

-Formalin-fixed, paraffin-embedded tissue block

OR

-2 Unstained, positively charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick

**Acceptable:** None

### Digital Image Access

1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: <https://news.mayocliniclabs.com/pathology/digital-imaging/>
2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: <https://news.mayocliniclabs.com/pathology/digital-imaging/#section3>

### Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

### Reject Due To

|   |        |
|---|--------|
| Wet/frozen tissue<br>Cytology smears<br>Nonformalin fixed tissue<br>Nonparaffin embedded tissue<br>Noncharged slides<br>ProbeOn slides<br>Snowcoat slides | Reject |
|---|--------|

### Specimen Stability Information

| Specimen Type | Temperature         | Time | Special Container |
|---------------|---------------------|------|-------------------|
| TECHONLY      | Ambient (preferred) |      |                   |
|               | Refrigerated        |      |                   |

### Clinical & Interpretive

#### Clinical Information

LIM domain only 2 (LMO2) is a transcription factor that regulates vascular and hematopoietic systems and is involved in

---

hematolymphoid neoplasia. LMO2 is preferentially expressed by germinal center B cells and may also be expressed in erythroid and myeloid precursors and in megakaryocytes. Expression has been observed in cases of lymphoblastic and acute myeloid leukemia. It is rarely expressed in mature T, natural killer, and plasma cell neoplasms and is absent from nonhematolymphoid tissues except for endothelial cells. In the diagnosis of B-cell lymphomas, LMO2 can be useful in an immunohistochemical panel to assign a germinal center phenotype.

**Interpretation**

This test does not include pathologist interpretation, only technical performance of the stain. If interpretation is required, order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

**Cautions**

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen dependent. Best practice is for paraffin sections to be cut within 6 weeks.

The charge of glass slides can be affected by environmental factors and subsequently may alter slide staining. Sending unsuitable glass slides can result in inconsistent staining due to poor slide surface chemistry.

Best practices for storage of positively charged slides:

- Minimize time slides are stored after being unpackaged
- Limit exposure to high humidity and heat
- Minimize exposure to plastics

**Clinical Reference**

1. Natkunam Y, Zhao S, Mason DY, et al. The oncoprotein LMO2 is expressed in normal germinal-center b cells and in human b-cell lymphomas. *Blood*. 2007;109(4):1636-1642
2. Blenk S, Engelmann J, Weniger M, et al. Germinal center b cell-like and activated b cell-like type of diffuse large b cell lymphoma: Analysis of molecular predictors, signatures, cell cycle state and patient survival. *Cancer Inform*. 2007;3:399-420
3. Natkunam Y, Farinha P, His ED, et al. LMO2 protein expression predicts survival in patients with diffuse large b -cell lymphoma treated with anthracycline-based chemotherapy with and without rituximab. *J Clin Oncol*. 2008;26(3):447-454
4. Gratzinger D, Zhao S, West R, et al. The transcription factor LMO2 is a robust marker of vascular endothelium and vascular neoplasms and selected other entities. *Am J Clin Pathol*. 2009;131(2):264-278
5. Younes SF, Beck AH, Lossos IS, et al. Immunoarchitectural patterns in follicular lymphoma: efficacy of HGAL and LMO2 in the detection of the interfollicular and diffuse components. *Am J Surg Pathol*. 2010;34(9):1266-1276
6. Magaki S, Hojat SA, Wei B, So A, Yong WH. An Introduction to the Performance of Immunohistochemistry. *Methods Mol Biol*. 2019;1897:289-298. doi:10.1007/978-1-4939-8935-5\_25

**Performance****Method Description**

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

1 to 3 days

**Specimen Retention Time**

Until staining is complete.

**Performing Laboratory Location**

Mayo Clinic Laboratories - Rochester Main Campus

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

**CPT Code Information**

88342-TC, primary

88341-TC, if additional IHC

**LOINC® Information**

| Test ID | Test Order Name     | Order LOINC® Value   |
|---------|---------------------|----------------------|
| LMO2    | LMO2 IHC, Tech Only | Order only;no result |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
|-----------|------------------|---------------------|

## Test Definition: LMO2

LIM Domain Only 2 (LMO2) Immunostain,  
Technical Component Only

|       |                     |                      |
|-------|---------------------|----------------------|
| 70813 | LMO2 IHC, Tech Only | Bill only; no result |
|-------|---------------------|----------------------|