

**Overview****Reflex Tests**

| Test Id | Reporting Name    | Available Separately | Always Performed |
|---------|-------------------|----------------------|------------------|
| FASCR   | Amphetamines, Rfx | No                   | No               |

**Testing Algorithm**

If the Amphetamine, Scr w-Conf, S is positive, then the Amphetamines, Rfx (FASCR) will be performed at an additional charge.

**Method Name**

Immunoassay (IA) â€” Screen

Gas Chromatography/Mass Spectrometry (GS/MS) â€” Confirmation; if applicable

**NY State Available**

Yes

**Specimen****Specimen Type**

Serum Red

**Specimen Required**

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 7 mL of serum refrigerated in a plastic vial.

**Specimen Minimum Volume**

3 mL

**Reject Due To**

|           |    |
|-----------|----|
| Hemolysis | NA |
| Lipemia   | NA |
| Icterus   | NA |
| Other     | NA |

**Specimen Stability Information**

| Specimen Type | Temperature | Time     | Special Container |
|---------------|-------------|----------|-------------------|
| Serum Red     | Ambient     | 72 hours |                   |

|  |                          |          |  |
|--|--------------------------|----------|--|
|  | Refrigerated (preferred) | 14 days  |  |
|  | Frozen                   | 180 days |  |

## Clinical & Interpretive

### Reference Values

Reference Range:

Amphetamines: Cutoff: 50

Confirmation Threshold: 10 mg/mL

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Sunday

### Report Available

5 to 9 days

### Performing Laboratory Location

Medtox Laboratories, Inc.

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### CPT Code Information

80307

80324, 80359-if applicable

### LOINC® Information

| Test ID | Test Order Name             | Order LOINC® Value |
|---------|-----------------------------|--------------------|
| FASCC   | Amphetamines, Scr w-Conf, S | 8149-7             |

---

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z4631     | Amphetamines     | 8149-7              |