

Overview

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FAMCS	Amphetamines Confirmation, S	No	No
FBCS	Benzodiazepines Confirmation, S	No	No
FCANS	Cannabinoids Confirmation, S	No	No
FCMC	Cocaine and Metabolites Confirm, S	No	No
FMMCS	Methadone and Metabolite Confirm, S	No	No
FF10S	Fentanyl/Acetyl Fentanyl Confirm, S	No	No
FOFU	Opiates-Free Unconjugated Conf, S	No	No
FPCCS	Phencyclidine Confirmation, S	No	No
FBCFS	Barbiturates Confirmation, S	No	No

Testing Algorithm

If appropriate, confirmation testing will be performed at an additional charge.

Method Name

Enzyme-Linked Immunosorbent Assay (ELISA)

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

**Specimen Type:** Serum

**Collection Container/Tube:** Red top

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 4 mL

**Collection Instructions:**

1. Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable.**

2. Centrifuge and send 4 mL of serum refrigerated in plastic, preservative-free vial.

**Specimen Minimum Volume**

1.75 mL

**Reject Due To**

Other	Polymer gel separation tube (SST or PST).
-------	---

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	14 days	

**Clinical & Interpretive**

**Reference Values**

Reporting limit determined each analysis.

Test	Result	Units
Opiates	None Detected	ng/mL
Cocaine/Metabolites	None Detected	ng/mL
Benzodiazepines	None Detected	ng/mL
Cannabinoids	None Detected	ng/mL
Amphetamines	None Detected	ng/mL
Barbiturates	None Detected	mcg/mL
Fentanyl/Acetyl Fentanyl	None Detected	ng/mL
Methadone/Metabolite	None Detected	ng/mL
Phencyclidine	None Detected	ng/mL
Methamphetamine/MDMA	None Detected	ng/mL
Oxycodone/Oxymorphone	None Detected	ng/mL

**Performance**

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 14 days

Performing Laboratory Location

NMS Labs

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

80307

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FD10S	Drugs of Abuse Screen, S	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z5245	Opiates	8219-8
Z5246	Cocaine/Metabolites	73823-7
Z5247	Benzodiazepines	46976-7
Z5248	Cannabinoids	8172-9
Z5249	Amphetamines	8149-7
Z5250	Barbiturates	20421-4
Z5251	Methadone/Metabolite	91033-1
Z5252	Phencyclidine	8236-2
Z5253	Methamphetamine/MDMA	91034-9
Z5254	Oxycodone/Oxymorphone	77732-6
Z6248	Fentanyl/Acetyl Fentanyl	29356-3