

## Overview

**Method Name**

Immunoassay (IA)

**NY State Available**

Yes

## Specimen

**Specimen Type**

Serum Red

**Specimen Required****\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*****Specimen Type:** Serum**Collection Container/Tube:** Red top**Submission Container/Tube:** Plastic vial**Specimen Volume:** 1 mL**Collection Instructions:**

1. Draw blood in a plain, red-top tube(s). **Serum gel tube is not acceptable.**
2. Centrifuge and send 1 mL of serum refrigerated in a plastic, preservative-free vial.

**Specimen Minimum Volume**

0.4 mL

**Reject Due To**

Other	Room Temperature; Serum gel tube (SST), Plasma gel tube (PST)
-------	---

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	90 days	

## Clinical & Interpretive

**Reference Values**

Reporting limit determined each analysis.

---

None Detected ng/mL

Therapeutic Range: 10 - 30 ng/mL

## Performance

### PDF Report

Referral

### Day(s) Performed

Monday - Sunday

### Report Available

9 to 11 days

### Performing Laboratory Location

NMS Labs

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### CPT Code Information

80299

### LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FDGTX	Digitoxin	3559-2

Result ID	Test Result Name	Result LOINC® Value
FDGTX	Digitoxin	3559-2