

Overview

Method Name

Immunoassay (IA)

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

Specimen Type: Serum

Collection Container/Tube: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Draw blood in a plain, red-top tube(s). Serum gel tube is not acceptable.

2. Centrifuge and send 1 mL of serum refrigerated in a plastic, preservative-free vial.

Specimen Minimum Volume

0.4 mL

Reject Due To

Other	Room Temperature; Serum gel tube (SST), Plasma gel tube (PST)
-------	---

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	90 days	

Clinical & Interpretive

Reference Values

Reporting limit determined each analysis.

None Detected ng/mL  
Therapeutic Range: 10 - 30 ng/mL

Performance

PDF Report  
Referral

Day(s) Performed  
Monday - Sunday

Report Available  
9 to 11 days

Performing Laboratory Location  
NMS Labs

Fees & Codes

- Fees
- Authorized users can sign in to [Test Prices](#) for detailed fee information.
  - Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
  - Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information  
80299

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FDGTX	Digitoxin	3559-2

Result ID	Test Result Name	Result LOINC® Value
FDGTX	Digitoxin	3559-2