

# **Test Definition: FFMTH**

Meconium Methadone Screen with Reflex Confirmation

# Overview

#### **Reflex Tests**

Test Id	Reporting Name	Available Separately	Always Performed
FMTHM	Meconium Methadone	No	No
	Confirm		

# **Testing Algorithm**

If screen is positive, then the Meconium Methadone Confirmation (FMTHM) will be performed at no additional charge.

#### **Method Name**

Immunoassay (IA); Chromatography/with Mass Spectrometry

# **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Meconium

# **Specimen Required**

Container/Tube: Sterile collection container

Specimen Volume: 3 g

**Collection Instructions:** Collect specimen into the same sterile collection container until 3 g of meconium have been collected or until the first milk stool appears. When at least 3 g of meconium has been collected, tightly screw on the cap of the collection vial and send specimen ambient.

Note: Specimens from different voids may be pooled if necessary.

# **Specimen Minimum Volume**

1 g

# Reject Due To

Hemolysis	NA NA
Lipemia	NA
Icterus	NA
Other	NA



# **Test Definition: FFMTH**

Meconium Methadone Screen with Reflex Confirmation

# **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Meconium	Ambient (preferred)	14 days	
	Refrigerated	14 days	
	Frozen	365 days	

# **Clinical & Interpretive**

# **Reference Values**

Initial presumptive testing by immunoassay at the following testing threshold:

Methadone: 50 ng/gm

Presumptive positive confirmed by definitive chromatography mass spectrometry (GC/MC or LC/MS-MS)

# **Performance**

# **PDF Report**

No

# Day(s) Performed

Monday through Sunday

# **Report Available**

1 to 6 days

# **Performing Laboratory Location**

Medtox Laboratories, Inc.

# **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

# **CPT Code Information**

80307



# **Test Definition: FFMTH**

Meconium Methadone Screen with Reflex Confirmation

# **LOINC®** Information

	Name O	Order LOINC® Value
FFMTH Meconium	Methadone Scr/Confirm 4.	13595-8

Result ID	Test Result Name	Result LOINC® Value
Z5603	Methadone	43595-8