

**Test Definition: FFMI2** 

Anti-Mi-2 Ab

# **Overview**

#### **Method Name**

RIPA Gel Radiography

#### **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Serum

# **Specimen Required**

**Collection Container/Tube:** 

**Preferred:** Red top **Acceptable:** Serum gel

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL serum

**Collection Instructions:** 

1. Within 1 hour of collection, centrifuge and aliquot 1 mL of serum into a plastic vial.

2. Send refrigerate.

# **Specimen Minimum Volume**

Serum: 0.5 mL Note: This volume does not allow for repeat testing.

# **Reject Due To**

Gross	Reject
hemolysis	
Gross lipemia	Reject
Gross icterus	Reject
Specimens	Reject
other than	
serum	
Bacterial	Reject
contamination	

# **Specimen Stability Information**

Specimen Type Temperature	Time	Special Container
---------------------------	------	-------------------



# **Test Definition: FFMI2**

Anti-Mi-2 Ab

Serum	Refrigerated (preferred)	14 days	
	Ambient	7 days	
	Frozen	60 days	

# **Clinical & Interpretive**

#### **Clinical Information**

Refer to www.labcorp.com/test-menu

#### **Reference Values**

Negative

#### **Performance**

#### **PDF Report**

Nο

# Day(s) Performed

Daily

#### **Report Available**

16 to 25 days

#### **Performing Laboratory Location**

**Esoterix Endocrinology** 

### **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **Test Classification**

This test was developed and its performance characteristics validated by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

# **CPT Code Information**

83516

# LOINC® Information



FFMI2

# **Test Definition: FFMI2**

18485-3

Anti-Mi-2 Ab

Test ID	Test Order Name	Order LOINC® Value
FFMI2	Anti-Mi-2 Ab	18485-3
Result ID	Test Result Name	Result LOINC® Value

Anti-Mi-2 Ab