

**Overview****Method Name**

RIPA Gel Radiography; Enzyme-Linked Immunosorbent Assay (ELISA)

**NY State Available**

Yes

**Specimen****Specimen Type**

Serum

**Specimen Required****Collection Container/Tube:** Red top or serum gel**Submission Container/Tube:** Plastic vial**Specimen Volume:** 3 mL Serum**Collection Instructions:**

1. Within 1 hour of collection, centrifuge and aliquot 3 mL of serum into a plastic vial.
2. Send refrigerated.

**Specimen Minimum Volume**

Serum: 1 mL; Note: This volume does not allow for repeat testing.

**Reject Due To**

Gross hemolysis:	Reject
Lipemia:	Reject
Icterus	Reject
Anything other than serum	Reject
Bacterial contamination	Reject

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Ambient	7 days	
	Frozen	60 days	

## Clinical & Interpretive

### Clinical Information

Refer to [www.labcorp.com/test-menu](http://www.labcorp.com/test-menu)

### Reference Values

Anti-PL-7 Ab, Anti-PL-12 Ab, Anti-EJ Ab, Anti-OJ Ab: Negative

Anti-Jo-1 Ab:

Negative: <20 units

Weak Positive: 20-39 units

Moderate Positive: 40-80 units

Strong Positive: >80 units

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Friday

### Report Available

16 to 25 days

### Performing Laboratory Location

Esoterix Endocrinology

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### Test Classification

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

### CPT Code Information

83516 x 4

86235

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FFRFT	Anti-Synthetase Profile	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z5782	Anti-Jo-1 Ab	35333-4
Z5783	Anti-PL-7 Ab	33772-5
Z5837	Anti-PL-12 Ab	33771-7
Z5823	Anti-EJ Ab	45149-2
Z5824	Anti-OJ Ab	45152-6