

# **Test Definition: FCSC**

Synthetic Cannabinoid Metabolites Screen, Expanded, Urine

### Overview

#### **Reflex Tests**

Test Id	Reporting Name	Available Separately	Always Performed
FCSCU	Synthetic Cannabinoid	No	No
	Confirm, U		

### **Testing Algorithm**

Confirmation test, FCSCU, will be performed, if appropriate, at no additional charge.

#### **Method Name**

High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

### **NY State Available**

Yes

## Specimen

## Specimen Type

Urine

### Specimen Required

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

Specimen Type: Urine

Container/Tube: Plastic preservative-free urine container

Specimen Volume: 5 mL

Collection Instructions: Collect 5 mL random urine specimen without preservative. Send specimen refrigerated in a

plastic preservative-free urine container.

Note: Known interference(s): FUBINACA 3,3-dimethylbutanoic acid: Quetiapine

## Specimen Minimum Volume

2.4 mL

## **Reject Due To**

Thawing:	Warm OK; Cold OK
----------	------------------

## **Specimen Stability Information**



# **Test Definition: FCSC**

Synthetic Cannabinoid Metabolites Screen, Expanded, Urine

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	30 days	
	Ambient	30 days	
	Frozen	180 days	

## **Clinical & Interpretive**

## **Reference Values**

Reporting limit determined each analysis.

<u>Test</u>	<u>Result</u>	<u>Units</u>
5F-PINACA 3-methylbutanoic acid	None Detected	ng/mL
4F-BINACA 3,3-dimethylbutanoic acid	None Detected	ng/mL
5F-PICA 3,3-dimethylbutanoic acid	None Detected	ng/mL
5F-PINACA 3,3-dimethylbutanoic acid	None Detected	ng/mL
MDMB-4en-PINACA butanoic acid	None Detected	ng/mL
FUBINACA 3-methylbutanoic acid	None Detected	ng/mL
FUBINACA 3,3-dimethylbutanoic acid	None Detected	ng/mL
4-carboxy-NA-PIM	None Detected	ng/mL

### **Performance**

## **PDF Report**

No

## Day(s) Performed

Monday through Sunday

## **Report Available**

10 to 16 days

## **Performing Laboratory Location**

**NMS Labs** 

## **Fees & Codes**

#### **Fees**

• Authorized users can sign in to <u>Test Prices</u> for detailed fee information.



# **Test Definition: FCSC**

Synthetic Cannabinoid Metabolites Screen, Expanded, Urine

- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

### **Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

### **CPT Code Information**

80307

#### **LOINC®** Information

	Test ID	Test Order Name	Order LOINC® Value
FCSC Synthetic		Synthetic Cannabinoid Screen, Urine	Not Provided
	Result ID	Test Result Name	Result LOINC® Value

Result ID	Test Result Name	Result LOINC® Value
Z6094	5F-PINACA 3-methylbutanoic acid	Not Provided
Z6095	4F-BINACA 3,3-dimethylbutanoic acid	Not Provided
Z6096	5F-PICA 3,3-dimethylbutanoic acid	Not Provided
Z6097	5F-PINACA 3,3-dimethylbutanoic acid	Not Provided
Z6098	MDMB-4en-PINACA butanoic acid	Not Provided
Z6099	FUBINACA 3-methylbutanoic acid	Not Provided
Z6100	FUBINACA 3,3-dimethylbutanoic acid	Not Provided
Z6101	4-carboxy-NA-PIM	Not Provided