

Overview

Method Name
Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

NY State Available
Yes

Specimen

Specimen Type
Serum Red

Specimen Required
Collection Container/Tube: Red top
Submission Container/Tube: Plastic vial
Specimen Volume: 3 mL serum
Collection Instructions:
1. Centrifuge and aliquot 3 mL of serum into a plastic vial.
2. Send refrigerate.

Specimen Minimum Volume
0.5 mL

Reject Due To

Gross Hemolysis	Reject
Gross Lipemia	Reject
Gross Icterus	Reject
Specimens other than serum collected in red top tube	Reject
Plasma or whole blood collected in tubes other than sodium heparin or	Reject

EDTA	
Serum or plasma gel tubes	Reject

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	180 days	

Clinical & Interpretive

Clinical Information

Refer to <https://medtox.labcorp.com/test-menu>

Reference Values

Reference Range: 1.0-10 ng/mL

Performance

PDF Report

No

Day(s) Performed

Tuesday through Sunday

Report Available

7 to 9 days

Specimen Retention Time

2 weeks

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80342

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
PROLX	Fluphenazine (Prolixin)	3650-9

Result ID	Test Result Name	Result LOINC® Value
Z1122	Fluphenazine	3650-9