

## Overview

### Method Name

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Varies

### Specimen Required

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top tube(s), **plasma gel tube is not acceptable**. Spin down and send 2 mL sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 2 mL of serum refrigerated in a plastic vial.

### Specimen Minimum Volume

0.3 mL

### Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	EDTA

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Ambient	72 hours	
	Refrigerated (preferred)	14 days	
	Frozen	180 days	

## Clinical & Interpretive

### Reference Values

Loxapine:

Reference Range: 5.0 - 30.0 ng/mL

8-Hydroxy-Loxapine:

Reference Range: 20.0 - 100.0 ng/mL

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Sunday

### Report Available

3 to 7 days

### Specimen Retention Time

2 weeks

### Performing Laboratory Location

Medtox Laboratories, Inc.

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### CPT Code Information

80342

### LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
---------	-----------------	--------------------

LOX	Loxapine (Loxitane)	17090-2
Result ID	Test Result Name	Result LOINC® Value
Z1125	Loxapine (Loxitane)	3728-3
Z1047	8-Hydroxy-Loxapine	9696-6