

## **Test Definition: FNAD**

Nadolol, Serum/Plasma

#### **Overview**

#### **Method Name**

High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

#### **NY State Available**

Yes

## **Specimen**

#### **Specimen Type**

Varies

#### Specimen Required

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*
Submit only 1 of the following specimens.

#### Serum

Specimen Type: Serum

Collection Container/Tube: Red top

Specimen Volume: 1 mL

**Collection Instructions:** Draw blood in a plain, red-top tube(s). (Serum gel tube is not acceptable.) Spin down and send 1 mL of serum refrigerated in a plastic preservative-free vial.

#### **Plasma**

Specimen Type: Plasma

Collection Container/Tube: Lavender top or pink top (EDTA)

Specimen volume: 1 mL

Collection Instructions: Draw blood in a lavender-top (EDTA) or Pink top tube(s). (Plasma gel tube is not acceptable.)

Spin down and send 1 mL of EDTA plasma refrigerated in a plastic preservative-free vial.

#### Specimen Minimum Volume

0.4 mL

#### Reject Due To

Other	Polymer gel separation tube (SST or PST)
-------	--

## **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
---------------	-------------	------	-------------------



# **Test Definition: FNAD**

Nadolol, Serum/Plasma

Varies	Refrigerated (preferred)	30 days	
	Ambient	30 days	
_	Frozen	30 days	

## **Clinical & Interpretive**

#### **Reference Values**

Reporting limit determined each analysis

None Detected ng/mL

Mean steady-state plasma levels following a daily regimen:

80 mg: 26 - 36 ng/mL 160 mg: 52 - 74 ng/mL 320 mg: 154 - 191 ng mL

#### **Performance**

#### **PDF Report**

No

#### Day(s) Performed

Monday through Sunday

#### Report Available

8 to 10 days

#### **Performing Laboratory Location**

**NMS Labs** 

#### **Fees & Codes**

## Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.



# **Test Definition: FNAD**

Nadolol, Serum/Plasma

## **CPT Code Information**

80299

## **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FNAD	Nadolol	12406-5

Result ID	Test Result Name	Result LOINC® Value
Z1430	Nadolol	12406-5
Z1867	Reporting Limit	19147-8