

## **Test Definition: FFLEX**

Cyclobenzaprine (Flexeril)

## Overview

#### **Method Name**

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

#### **NY State Available**

Yes

## Specimen

### **Specimen Type**

Varies

## **Specimen Required**

Submit only 1 of the following specimens:

#### **Plasma**

Specimen Type: Plasma Container/Tube: Green Top Specimen Volume: 2 mL

Collection Instructions: Draw blood in a green-top (sodium heparin) tube, plasma gel tube is not acceptable. Spin down

and send 2 mL of sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Specimen Type: Serum Container/Tube: Red Specimen Volume: 2 mL

Collection Instructions: Draw blood in a plain, red-top tube, serum gel tube is not acceptable. Spin down and send 2 mL

of serum refrigerated in a plastic vial.

## **Specimen Minimum Volume**

0.4 mL

## **Reject Due To**

Hemolysis	NA NA
Lipemia	NA NA
Icteric	NA
Other	NA



# **Test Definition: FFLEX**

Cyclobenzaprine (Flexeril)

## **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Ambient	72 hours	
	Frozen	30 days	

## **Clinical & Interpretive**

#### **Reference Values**

Reference Range: 10 - 30 ng/mL

#### **Performance**

## **PDF Report**

No

## Day(s) Performed

Monday through Sunday

## **Report Available**

3 to 7 days

## **Specimen Retention Time**

2 weeks

## **Performing Laboratory Location**

Medtox Laboratories, Inc.

#### **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

## **CPT Code Information**

80369



# **Test Definition: FFLEX**

Cyclobenzaprine (Flexeril)

## **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FFLEX	Cyclobenzaprine (Flexeril)	6797-5

Result ID	Test Result Name	Result LOINC® Value
Z1130	Cyclobenzaprine (Flexeril)	6797-5