

Overview**Method Name**

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

NY State Available

Yes

Specimen**Specimen Type**

Varies

Specimen Required

Submit only 1 of the following specimens:

PlasmaDraw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 2 mL sodium heparin plasma frozen in plastic vial.**Serum**Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 2 mL of serum frozen in plastic vial.**Specimen Minimum Volume**

0.3 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Frozen	180 days	

Clinical & Interpretive

Reference Values

Flurazepam:

Reference Range: 0 - 30 ng/mL

Desalkylflurazepam:

Reference Range: 30 - 150 ng/mL

Flurazepam + Desalkylflurazepam:

Reference Range: 30 - 180 ng/mL

Performance**PDF Report**

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 9 days

Specimen Retention Time

2 weeks

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80346

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
---------	-----------------	--------------------

FFLUR	Flurazepam (Dalmane)	73826-0
-------	----------------------	---------

Result ID	Test Result Name	Result LOINC® Value
Z1133	Flurazepam	46979-1
Z1160	Desalkylflurazepam	46980-9
Z1161	Flurazepam + Desalkylflurazepam	73826-0