

Test Definition: FAZAT

Azathioprine (Imuran) as 6-Mercaptopurine

Overview	
	١

Method Name

High Performance Liquid Chromatography with Ultraviolet Detection (HPLC-UV)

NY State Available

Yes

Specimen

Specimen Type

Varies

Specimen Required

Submit only 1 of the following specimens:

Plasma

Draw blood in a green-top (sodium heparin) tube(s), plasma gel tube is not acceptable. Spin down and send 2 mL of sodium heparin plasma refrigerated in a plastic vial.

Serum

Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable.** Spin down and send 2 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.3 mL

Reject Due To

Hemolysis	NA NA
Lipemia	NA
Icterus	NA NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	180 days	



Test Definition: FAZAT

Azathioprine (Imuran) as 6-Mercaptopurine

Clinical & Interpretive

Reference Values

Units: ng/mL

Azathioprine is measured as the metabolite, 6-mercaptopurine. Therapeutic and toxic ranges have not been established. Usual therapeutic doses produce 6-mercaptopurine serum concentrations of less than 1000 ng/ml.

Performance

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 9 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

CPT Code Information

80299

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FAZAT	Azathioprine (Imuran)	16419-4
	•	
Result ID	Test Result Name	Result LOINC® Value