

## Overview

### Method Name

Gas Chromatography-Mass Spectrometry (GC-MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Varies

### Specimen Required

Submit only 1 of the following specimens:

#### Serum

Draw blood in a plain red-top tube(s), serum gel tube is not acceptable. Spin down and send 5 mL of serum refrigerated in a plastic vial.

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), plasma gel tube is not acceptable. Spin down and send 5 mL sodium heparin plasma refrigerated in a plastic vial.

### Specimen Minimum Volume

2 mL

### Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Frozen	180 days	

## Clinical & Interpretive

### Reference Values

Toxic: Greater than 100 ng/mL

Serious Toxicities likely: Greater than 300 ng/mL

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Sunday

### Report Available

5 to 9 days

### Specimen Retention Time

2 weeks

### Performing Laboratory Location

Medtox Laboratories, Inc.

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### CPT Code Information

83992

### LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
PCPUG	Phencyclidine (PCP), Serum	3934-7

Result ID	Test Result Name	Result LOINC® Value
Z1051	Phencyclidine	3934-7